



Medicator Testing

Water medicators must be tested before each use with medication. This is important to ensure you are administering the right dose of medication to your birds.

1 Disconnect the outflow side of the medicator from the water line.



2 Fill a measuring cup with a specific amount of water.



3 Place a pail under the outflow of the medicator.



4 Place the medicator intake tube in the measuring cup.



5 Turn on the water supply through the medicator.



6 If the correct amounts are disappearing out of the measuring cup, then the medicator is working properly. If not, your medicator needs servicing (i.e. for medicators set at 1%, when 10 L has collected in the pail, at the outflow end, there should be 100 mL gone from the measuring cup).



For additional information, **check out** this how-to video at <https://bit.ly/2xAevSu>